


### **Buttons and connections**

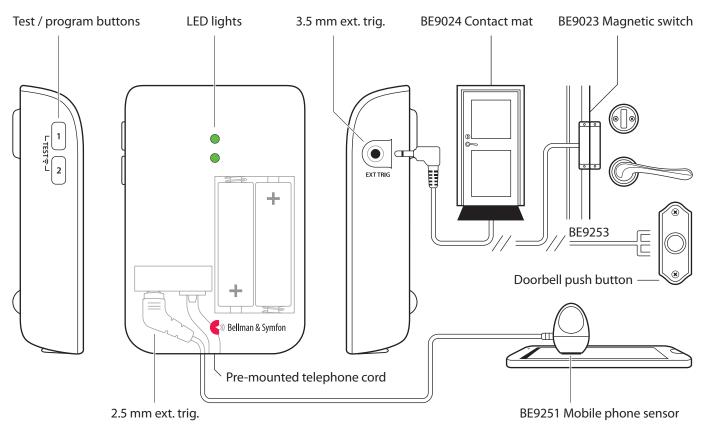

# **Technical specifications**

#### In the box

- BE1431 Visit telephone transmitter
- 2 x 1.5 V AA alkaline batteries
- Telephone cord and adapter
- Screw and wall plug

### **Power and battery**

- Battery power
   2 x 1.5 V AA lithium or alkaline
   type batteries
- Power consumption Active < 70 mA Idle position < 15 μA</li>
- Operation time
   Alkaline batteries ~ 5 years
   Lithium batteries ~ 10 years

#### **Dimensions and weight**

- Height: 100 mm, 4.0"
- Width: 65 mm, 2.6"
- Depth: 27 mm, 1.1"
- Weight: 120 g, 4.2 oz. incl. batteries

#### **Activation**

- The test buttons
- A landline telephone
- A smartphone or tablet via the mobile phone sensor
- A contact mat or magnetic switch
- A doorbell connected to the ext trig

## **Environment**

For indoor use only

### Accessories

- BE9251 Mobile phone sensor
- BE9023 Magnetic switch
- BE9024 Contact mat
- BE9253 Ext. trig. cable, 3.5 mm

### Inputs


- RJ11 analogue telephone input
- 2.5 mm external trigger input
- 3.5 mm external trigger input

#### Frequency and coverage

- Frequency: 314.91 MHz, 433.92 MHz or 868.30 MHz, depending on region
- Coverage: 50 250 m, 55 273 yd. depending on the radio frequency and the building's characteristics

## Setting up the transmitter

- 1 Remove the battery pull tab to start the unit.
  Clean the wall with the wet wipe and remove the protective film from the Velcro.
- 2 Mount the transmitter on the wall. You can also use the supplied screw and plug.
- **3** Connect the telephone adapter as shown below.

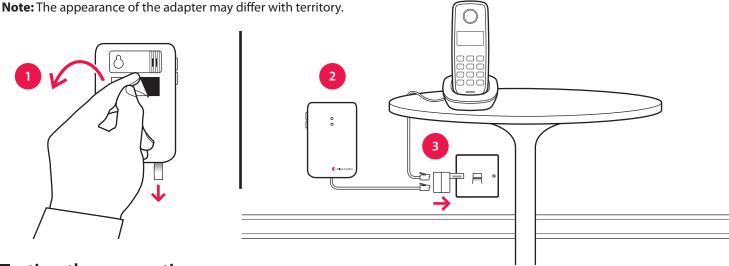

## **Testing the connection**

#### Using the test button


- 1 Press both test buttons simultaneously on the telephone transmitter. The top LED lights up in green to show that a radio signal is being transmitted.
- 2 The yellow Visit LED on the receiver lights up to show that the signal was received. In addition it starts to sound, flash or vibrate depending on the receiver.


**Note:** If you have changed the signal pattern, it will react in accordance with the table on the following spread.

#### Using a mobile phone

- 1 Use for instance a mobile phone to call the landline telephone. The top LED on the transmitter lights up in green to show that an incoming call is detected.
- 2 The yellow Visit LED on the receiver lights up to show that the radio signal was received. In addition it starts to sound, flash or vibrate depending on the receiver.

**Note:** If you have changed the signal pattern, it will react in accordance with the table on the following spread.

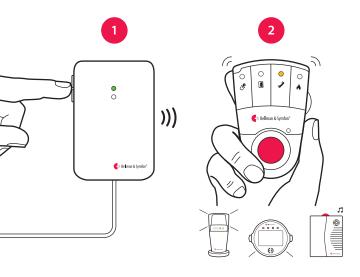

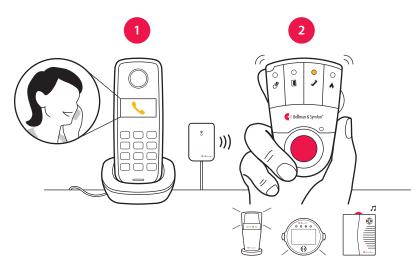

## **Default signal pattern**

When the telephone transmitter is activated by an incoming call or a triggered accessory, the following happens:

- 1 The LED on the transmitter lights up to show that it's signalling the receiver.
- 2 The Visit LED on the receiver lights up and it starts to sound, flash or vibrate with a certain pace, called signal pattern. The transmitter and the connected accessories determine the signal pattern. The default is as follows:

#### **Transmitter**

#### | Receiver signal pattern

Source	LED	LED	Sound	Vibration	Flash
<ul><li>Landline phone</li></ul>	Green, top	Yellow light	1 x ring signal, low	Medium ■□■□	Yes
Mobile phone sensor	Green, top	Yellow blinks	2 x ring signal, high	Medium ■□■□	Yes
<ul><li>Other accessory</li></ul>	Green, bottom	Green light	1 x door chime, low	Slow ■□□□	Yes

## Changing the signal pattern

Other accessory

The transmitter controls the signal pattern. Open the transmitter front cover and move the signal switches according to the table below to change it:



#### **Transmitter**

#### Receiver signal pattern

Switch	Source	LED	Sound	Vibration	Flash
1 2 3 4	Landline phone / test button Mobile phone sensor Other accessory	Yellow light Yellow blinks Green light	1 x ring signal, low 2 x ring signal, high 1 x door chime, low	Medium BOBO	Yes Yes Yes
1 2 3 4	Landline phone / test button Mobile phone sensor Other accessory	Yellow light Yellow blinks 2 x green blinks	1 x ring signal, low 2 x ring signal, high 2 x door chime, low	Medium BOBO	Yes Yes Yes
1 2 3 4	Landline phone / test button Mobile phone sensor Other accessory	Yellow light Yellow blinks 3 x yellow blinks	1 x ring signal, low 2 x ring signal, high 1 x ring signal, high	Medium ■□■□ Medium ■□■□ Medium ■□■□	Yes Yes Yes
1 2 3 4	Landline phone / test button Mobile phone sensor Other accessory	Yellow light Yellow blinks 2 x orange blinks	1 x ring signal, low 2 x ring signal, high Baby melody	Medium BOBO	Yes Yes Yes
1 2 3 4	Landline phone / test button Mobile phone sensor Other accessory	2 x yellow blinks Yellow light 3 x orange blinks	2 x ring signal, low 1 x ring signal, low Baby melody	Fast 10101010 Medium Tast 10101010	Yes Yes Yes
1 2 3 4	Landline phone / test button Mobile phone sensor Other accessory	2 x yellow blinks Orange blinks Green blinks	2 x ring signal, low Baby melody 2 x door chime, high	Medium Fast INTOINED	Yes Yes Yes
1 2 3 4	Landline phone / test button Mobile phone sensor Other accessory	Orange blinks 3 x yellow blinks 2 x green blinks	Baby melody 1 x ring signal, high 2 x door chime, low	Medium ■□■□ Medium ■□■□ Slow ■□□□	Yes Yes Yes
Ш	Landline phone / test button Mobile phone sensor	3 x yellow blinks 2 x yellow blinks	1 x ring signal, high 2 x ring signal, low	Medium ■□■□ Medium ■□■□	Yes Yes

Green blinks

2 x door chime, high

Slow

Yes


## Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference you need to change the radio key on all units. The radio key switches are located under the transmitter cover.

### Here is how you change the radio key:

- 1 Open the transmitter front cover and move any radio key switch to the up = on position to change the radio key. (By default, all radio key switches are positioned down = off.)
- 2 Press and hold the test/function button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- **3** Press both test buttons simultaneously on the transmitter within 30 seconds to send the new radio key.
- 4 All Visit LEDs on the receiver blink 5 times to show that the radio key has been changed. It then returns to normal mode.

**Note:** All Visit units must be set to the same radio key in order to operate as a group.

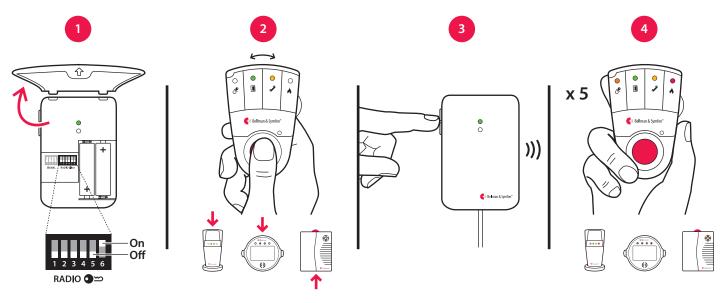

## **Troubleshooting**

If	Try this
The LEDs blink in orange every minute	<ul> <li>Replace the batteries. Only use 1.5 V AA (LR6) lithium or alkaline batteries.</li> </ul>
The transmitter LED lights up in green but the receiver doesn't respond	<ul> <li>Check the transmitter batteries and the receiver batteries and connections.</li> <li>Move the receiver closer to the transmitter to make sure it's within radio range.</li> <li>Check that the units are set to the same radio key, see Changing the radio key.</li> </ul>
The transmitter LED doesn't light up when the phone rings or when an accessory is triggered	<ul> <li>Press the test buttons on the transmitter. If the LED lights up in green, check all connections. If the LED doesn't light up in green, replace the batteries. Only use 1.5 V AA (LR6) lithium or alkaline batteries.</li> </ul>
The transmitter LED doesn't light up when I press the test buttons	<ul> <li>Replace the batteries. Only use 1.5 V AA (LR6) lithium or alkaline batteries. If the LED still doesn't light up, contact your retailer for service information.</li> </ul>
The receiver is activated for no apparent reason	<ul> <li>There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see Changing the radio key.</li> </ul>